






Note before use

- Only licensed medical practitioners are allowed to use this product, or it shall be used under their guidance in person.
- Before use, operators should fully understand the product information and instructions.
- The company is not responsible for injury, infection or other damage caused by improper storage, preparation, use or disposal of electrodes.

1. Disposable Subdermal Needle Electrodes






1) The labeling carries the following information:

	Do not re-use
	Use by date
	Do not use when package is broken
	Sterilized using ethylene oxide
	Type BF applied part

- 2) Good electrical signal transmission.
- 3) Biocompatibility Risk $P < 10^{-6}$.
- 4) Infection Risk $P < 10^{-6}$.
- 5) Dispose the needles immediately after use in a medical waste container to avoid injuring operators and other personnel. Dispose the electrodes according to hospital, administrative, or government policies. To avoid cross-infection, do not reuse the needles.

2. Disposable Nerve Stimulator Probes


1) The labeling carries the following information:





	Do not re-use
	Use by date
	Do not use when package is broken
	Sterilized using ethylene oxide
	Type BF applied part

- 2) Good electrical signal transmission.
- 3) Biocompatibility Risk $P < 10^{-6}$.
- 4) Infection Risk $P < 10^{-6}$.
- 5) Dispose the electrodes according to hospital, administrative, or government policies. To avoid cross-infection, do not reuse the probes.

3. Disposable EMG Needle Electrodes

1) The labeling carries the following information:




	Do not re-use
---	---------------

	Use by date
	Do not use when package is broken
	Sterilized using ethylene oxide
	Type BF applied part

- 2) Good electrical signal transmission.
- 3) Biocompatibility Risk $P < 10^{-6}$.
- 4) Infection Risk $P < 10^{-6}$.
- 5) Dispose the needles immediately after use in a medical waste container to avoid injuring operators and other personnel. Dispose the electrodes according to hospital, administrative, or government policies. To avoid cross-infection, do not reuse the needle.

4. Disposable Pre-Gelled Surface Electrodes




1) The labeling carries the following information:

	Do not re-use
	Use by date
	Do not use when package is broken

- 2) Good electrical signal transmission.
- 3) Reuse of single use electrodes increases the risk of infection and may cause degraded or ineffective monitoring.
- 4) Dispose the electrodes according to hospital, administrative, or government policies.

5. Medical Extension Lead Wires and Cables

1) The labeling carries the following information:

	Do not re-use
	Use by date
	Do not use when package is broken

- 2) Good electrical signal transmission.
- 3) When the product cannot be reused, dispose the products according to hospital, administrative, or government policies.